


FLNT Gloss

Resin Base Grinding Tool Kit



FLNT Gloss Resin Base Grinding Tool Kit is a post-processing system for resin denture base restorations, composed of grinding tools in various geometries and grit sizes. With high grinding strength and excellent wear resistance, it enables efficient extraoral adjustment and finishing of dental resin base restorations.

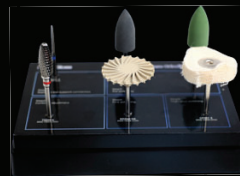

6pcs/kit

Advantages

- All-in-one extraoral grinding and polishing kit for resin denture base restorations.
- The large-size design increases the contact area, thereby improving grinding efficiency.
- The rapid chip-evacuation and heat-dissipation structure effectively reduces surface temperature, preventing damage to restorations.
- Rubber cyclone wheels and cloth buff wheels, combined with polishing paste, enable efficient surface refinement and high-gloss mirror finish.

Technical Data

Step 1

Separating the support connectors of the restoration



Model: TH201 3

Recommended Speed:
10,000 rpm

Step 2

Shaping and contouring of the restoration



Model: TI209 4

Recommended Speed:
8,000-10,000 rpm

Step 3

Surface smoothing and pre-polishing



Model: SK214 2



Model: SK214 3

Recommended Speed:
20,000-30,000 rpm

Step 4

Fine polishing of the restoration surface



Model: RD240 8A

Recommended Speed:
6,000-10,000 rpm

Step 5

High-gloss polishing of the restoration surface



Model: CF201 2

Recommended Speed:
6,000-10,000 rpm