

FLNT Gloss

Resin C&B Grinding Tool Kit



FLNT Gloss Resin C&B Grinding Tool Kit is a post-processing system for PMMA resin crown and bridge restorations, composed of grinding and polishing tools in geometries and abrasive formulation. With high grinding strength and excellent wear resistance, it enables efficient extraoral adjustment, finishing, and polishing of dental resin restorations.



7pcs/kit

Advantages

- A complete kit for the extra-oral grinding and polishing of resin crown and bridge restorations.
- Manufactured from high-grade special tungsten carbide that receives an enhanced surface treatment, guaranteeing extended cutting sharpness and longevity.
- The optimized design for chip evacuation and heat management effectively minimizes temperature rise at the interface, preventing thermally induced damage to the restoration.
- Rubber cyclone wheels and cloth buff wheels, combined with polishing paste, deliver efficient surface refinement and a brilliant mirror finish.

Technical Data

Step 1

Separating the support connectors of the restoration



Model: TH201 3



Model: DD002 4A

Recommended Speed:
10,000 rpm

Step 2

Shaping and contouring of the restoration



Model: TI207 4

Recommended Speed:
15,000-20,000 rpm

Step 3

Surface smoothing and pre-polishing



Model: SK213 2



Model: SK213 5

Recommended Speed:
20,000-30,000 rpm

Step 4

Fine polishing of the restoration surface



Model: RD240 8A

Recommended Speed:
10,000-15,000 rpm

Step 5

High-gloss polishing of the restoration surface



Model: CF201 2

Recommended Speed:
8,000-12,000 rpm